


EXCEPT...

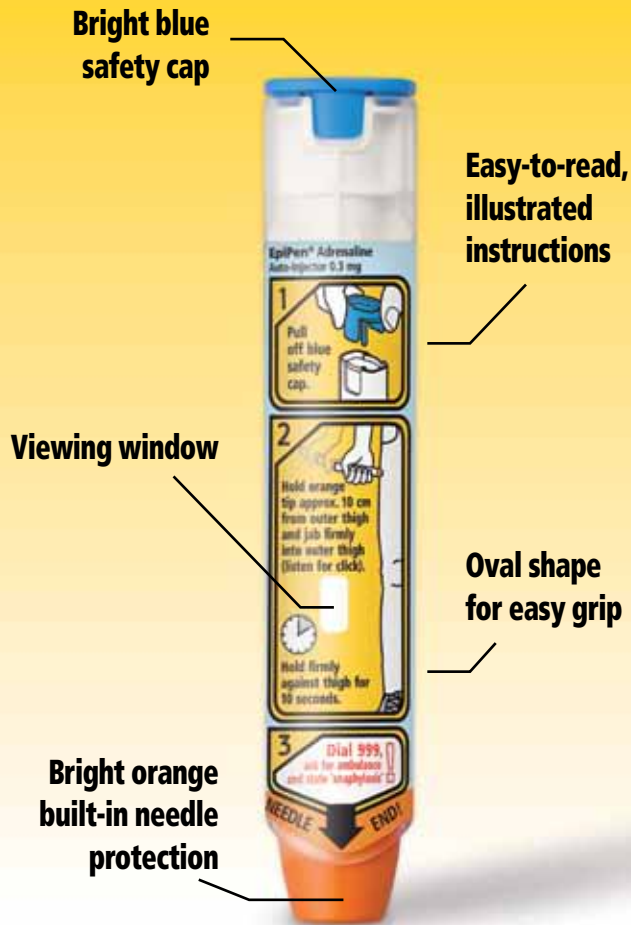

Comes with a free hard carry case

EPIPEN® EPIPEN® Jr
(Adrenaline) Auto-Injectors 0.3/0.15mg

ADDED BENEFITS

Colour-coded ends

to avoid confusion - remember 'blue to sky, orange to thigh'

Clear instructions

for you or another person to use in an emergency situation

Viewing window

lets you check the solution is clear and that the injection has been given



Orange needle guard

keeps the needle covered during and after use



IMPORTANT INFORMATION

If your symptoms don't improve after the first injection, **you can give a second one 5-15 minutes later.**

Ask your doctor if you are worried about this as you may need to carry two pens.

Carry your EpiPen® (adrenaline) auto-injectors with you at all times.

Practise with your trainer pen (see back page for details).

Check the viewing window regularly to make sure the solution is clear.

If it isn't, **replace your EpiPen®** (adrenaline) auto-injector as soon as possible.

Make sure your pen isn't out of date – sign up for our Expiry Alert Service (see back page for details).

SAME FAMILIAR ADMINISTRATION TECHNIQUE

Step 1

Lie down with your legs slightly elevated or sit up if breathing is difficult

Step 2



Grasp your **EpiPen**® (adrenaline) auto-injector in your dominant hand with the blue safety cap closest to your thumb and remove cap

Step 3



Hold your **EpiPen**® (adrenaline) auto-injector about 10 cm away from your leg and **swing and jab** the orange tip into the upper outer thigh at a 90° angle

Hold in place for 10 seconds

Remove the **EpiPen**® (adrenaline) auto-injector and massage injection site for at least 10 seconds

Step 4



You must dial 999, ask for an ambulance and **state 'anaphylaxis'** (pronounced 'anna-fill-axis') immediately after administering the **EpiPen**® (adrenaline) auto-injector

Do not get up, stay lying down or seated until you have been assessed by a paramedic

UK/EPI/12/0075 August 2012

EXPIRY ALERT SERVICE

YOU CAN REGISTER YOUR **EPIPEN**® (ADRENALINE) AUTO-INJECTOR AT WWW.EPIPEN.CO.UK TO RECEIVE FREE EXPIRY ALERTS

VISIT WWW.EPIPEN.CO.UK TO:

- ✓ FIND OUT MORE ABOUT ANAPHYLAXIS
- ✓ ORDER A FREE TRAINER PEN*
- ✓ REQUEST HELPFUL LEAFLETS/FORMS
- ✓ VIEW A SHORT DEMONSTRATION VIDEO



EPIPEN® **EPIPEN**® Jr
(Adrenaline) Auto-Injectors 0.3/0.15mg

*Postage and packaging to be paid

WE'VE CHANGED NOTHING



New information for patients who have been recently prescribed an **EpiPen**® (adrenaline) auto-injector

EPIPEN® **EPIPEN**® Jr
(Adrenaline) Auto-Injectors 0.3/0.15mg

Provided as a service to patient education by MEDA Pharmaceuticals Ltd